Create it 🖌

MOBILE DENTISTRY



MOBILE DENTISTRY

Advanced Dental Care for Everybody.

We all know that tooth decay is treatable. But even in today's world, there are still regions without access to dental care. In areas where dentists are available it is not always easy to visit them in the clinic-especially for the elderly and less mobile. NSK has designed a portable treatment unit that has the functionality of a fully equipped treatment unit to help deliver advanced dental care regardless of the patients' location. Through various innovative ideas NSK's mobile dentistry is leading the way in providing mobile dental care.

INDEX

- Dentalone
- VIVA ace
- VIVAMATE G
- Related Produ
- Spare Parts a

	P. 03 - 04
	P. 05 - 08
5	P. 09 - 10
lucts	P. 11 - 12
and Accessories	P. 13 - 14

Dentalone Mobile Dental Cart

Helping You Care. Anywhere.*

Dentalone is the dentist's office mobile cart which is provided for restorative or preventive dentistry in the field such as Humanitarian, Home Care, Schools, and Army & Hospitals.

Advantages

- Compact
- Simple to use
- Mobile
- Plug and Play system
- Easy to carry
- User-friendly
- Extends up from case to cart
- Large range of functionality

Features

- All components pack into a case with an extendable carrying handle
- Up and down function of control unit and instrument carrier which is barrier fully for treatment position • Small, light, complete unit
- Instrument carrier with automatic selection of instruments and easy positioning
- Integrated compressor for irrigation & 3-in-1 syringe • Universal voltage adaptability (100 V-230 V)



NLX nano **Clinical Electric Micromotor** Smaller and lighter micromotor currently available.

Varios2 Lux

Multi Function Ultrasonic Scalers iPiezo engine ensures stable power output and performance.

- Illuminated LCD control panel Multi Pad
- 40,000 min⁻¹ autoclavable brushless NLX nano micromotor with LED light, internal water and titanium body
- Integrated multi function ultrasonic scaler Varios 170 with LED light, slim and lightweight VA2-LUX-HP handpiece
- 3-way air / water / spray syringe with LED light
- Saliva complete suction
- 500 mL water tank
- 1 Liter waste container



Dentalone Complete Set

MODEL	ORDER CODE
Dentalone	S9070

Contents

• 1 Carrying Case • 1 Water tank for spray • 1 Tank for vacuum • Built in compressor Built in suction pump • 1 Variable Foot Control 1 Extendable Tablet

lv required

MEK Dension 2000000 Med Annum 1 State 1 State
 elinelinelin X. Honesen Y.

Multi Pad Multi-Control Panel for built-in system

Visual identification on the display is possible even during operation of the handpiece.



Specifications

[Dimensions]

- Case with trolley (unpulled trolley handle): W 351 x D 405 x H 545 mm Case with trolley (pulled trolley handle): W 351 x D 405 x H 1,075 mm
 Without case : W 350 x D 405 x H 496 mm
 Working height : 766 mm
- Weight with case : 18.5 kg
- Power Supply : AC 100-230 V 50/60 Hz

VIVA ace Mobile Dentistry System

Wherever Possible

For more comfortable & more advanced home visits for dental treatment

As societies age more rapidly, there is an ever greater need for mobile dental treatment. Today's ageing population keeps more of their natural teeth, which means a wider range of treatments is required nearer to the patient. VIVA ace combines the functionality of a dental treatment centre in a compact body with multipurpose micromotor, ultrasonic scaler and more. Its lightweight build means it is easy to carry and straight forward to set up in different locations. The first of its kind with high-functionality, supports more comfortable dental treatment wherever the patient.



(High Funcitionality, Just Like a Treatment Centre

Easy to Set-up and Breakdown





Lightweight and Compact Body

Highly manoeuvrable and lightweight, weighing only 8.6 kg. Its compact size is ideal for stowing in a car, is easily carried up and down stairs and fits into compact spaces. Simple to transport from one patient to another. The small size has been achieved by making each component lightweight and compact without compromising functionality.





Photos are of optional extras



Advanced Functionality in a **Compact Body**

Fresh water tank



Foot controls





Highly Functional Portable Dental System Just Like a Treatment Centre

Excellent efficiency and safety are achieved via both the intuitive and highly visible large LCD control panel and the superb functionality provided by each individual instrument. A high functionality system, just like a treatment centre.

High Functionality Micromotor

Full endodontic functionality



NLX plus

Ideally suited for all state of the art dental requirements

	Rotation speed (min-1)	Torque (Ncm)
General mode	1,000-40,000	4.0
Endodontic mode	100-5,000	0.3-3.0

3 in 1 syringe



Lightweight and simple 3 in 1 syringe operation. The syringe nozzle uses a simple quick release mechanism.

Foot controls

For treatments performed by just one practitioner

The micromotor, ultrasonic scaler and suction can be switched ON/OFF using one foot control in Sync mode.



Micromotor, Scaler & Suction

Multipurpose Ultrasonic Scaler

Comprehensive tip range





NSK iPiezo ultrasonic engine Provides the appropriate power at the tip end adapting to tooth conditions for firm or soft tartar. Safe scaling is possible even when removing hard tartar.


Suction



Built in ball leaver valve to adjust suction power.

For treatment performed by 2 operators, 1 dentist/hygienist and 1 dental nurse

Switch micromotor, ultrasonic scaler and suction ON/OFF using two foot controls. The two in combination lets the dentist concentrate on operating the micromotor and scaler.

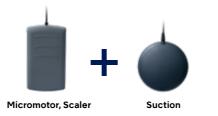



Quick Set up and Storage

Set up and storage is straight forward, simple and quick, making work more efficient and easier. The instrument holders and arm can be set at 4 different levels.

Hygienic as the components that touch the floor are kept in a separate storage compartment with a cover







Storage is simple Remove the handpieces for easy storage.

Very easy to maintain

Light Probe (optional)

Bright illumination of the oral cavity in any treatment environment. Good accessibility with a slim body that also makes it possible to access hard-to-see areas.





VIVA ace Complete Set

MODEL

ORDER CODE

VIVA ace Complete Set

Y1004228

Contents • VIVA ace Basic Set • VIVA ace Motor Kit • VIVA ace Scaler Kit

Specifications (Control Unit)

- Dimensions : W 462 × D 298 × H 284 mm
- Unit weight : 8.6 kg
 Power supply : AC 100-240 V 50/60 Hz
- Power source input : 130 VA

VIVAMATEG5 Portable Micromotor Unit

Portable LED Micromotor; from Denture Adjustment to **General Applications**

The latest member of pioneering VIVAMATE portable micromotor series, VIVAMATE G5 provides various functions to meet mobile dental needs. The newly developed control unit with innovative design offers portability and operability by being both mains and battery powered.

-vnvamare65

0

Control Panel Offers Simple and Intuitive Operation

Easy to see necessary information, such as rotation speed setting, power on/off and battery remaining levels. The control panel that allows intuitive operation greatly reduces stress to operate the instrument.

LED Micromotor

The compact size micromotor has been specifically developed for mobile dentistry and weights only 148g (w/cord) and features an LED light. Although fully portable it performs like an in-surgery system meeting a wide range of clinical requirements and is compatible with your existing contra-angles and straight handpieces^{*1} helping you to manage the cost of a mobile dentistry set-up.

* 1 Conformable to ISO 3964.



Use the built-in rechargeable battery or plug into the mains.

Able to operate with rechargeable batteries anywhere. When battery power level is low, just plug in AC adapter into outlet; bring the unit to patient visits without worry. It is very easy to carry since the control unit is palm-size (W 72 x D 180 x H 36 mm) and weighs only 370g.

Foot Control (optional)

Conveniently adjust the motor speed by



depressing the pedal of the foot control without taking your eyes of the treatment.





* 2 Including Motor Cord

Carrying Case

Comes with a carrying case designed for storing VIVAMATE G5 unit and its attachments.



VIVAMATE G5 Complete Set

MODEL

VIVAMATE G5 230 V

ORDER CODE

Y1002801

- Contents
 Control unit Handpiece Stand Motor (Including Motor Cord)
- AC Adaptor
 AC Power Cord
 Carrying Case

Specifications

- [Control Unit] Dimensions : W 72 x D 180 x H 36 mm • Weight : 370 g
- Charge/Continuous use : 2 h / 1 h (depending on workload)
- [Micromotor]
- Dimensions : ø22 x L 68.6 mm
- Weight : 148 g (Include Motor cord)
 Speed : 3,000-30,000 min⁻¹
- Torque : 3 Ncm (30,000 min⁻¹ while rotating)

Related Products

Contra-angle







NSK Ti-Max X65L

For General Application

Gear Ratio	Optic	MODEL	ORDER CODE
1:5 Increasing	٠	Z95L	C1034

 Max Speed : 200,000 min⁻¹
 For FG burs (ø1.6)
 Push Button Chuck Titanium Body with Scratch Resistant DURAGRIP • Ceramic Bearings
 Quattro Spray (with two way spray function) • Microfilter
 Anti Heat System • Clean Head System • Cellular Glass Optics

For General Application

Gear Ratio Optic MODEL ORDER CODE	1:1 Direct Drive	٠	Z25L	C1038
	Gear Ratio	Optic	MODEL	ORDER CODE

 Max Speed : 40,000 min⁻¹ • For CA burs (ø2.35)
 Titanium Body with Scratch Resistant DURAGRIP • Push Button Chuck Single Spray
 Microfilter
 Clean Head System
 Cellular Glass Optics

For Prophylaxis Application

Gear Ratio	Optic	MODEL	ORDER CODE
16:1 Reduction	۲	X57L	C609

Max Speed : 2,500 min⁻¹
 For Screw-in Cups and Brushes

 360° Rotation
 •w/o Spray
 • Special Dust Proof

 • Titanium Body with Scratch Resistant DURACOAT
 • Cellular Glass Optics


Straight Handpiece

1:1 Direct Drive	•	X65L	H273
Gear Ratio	Optic	MODEL	ORDER CODE

• Max Speed : 40,000 min⁻¹ • For HP burs (ø2.35) • For CA burs (ø2.35)* Titanium Body with Scratch Resistant DURACOAT • Single Spray
 Clean Head System • Cellular Glass Optics *Need to insert the supplied bur stopper

Ultrasonic Scaler Tip

NSK Offers a Wide Choice of Varios Ultrasonic Scaler Tips for Clinical Applications.

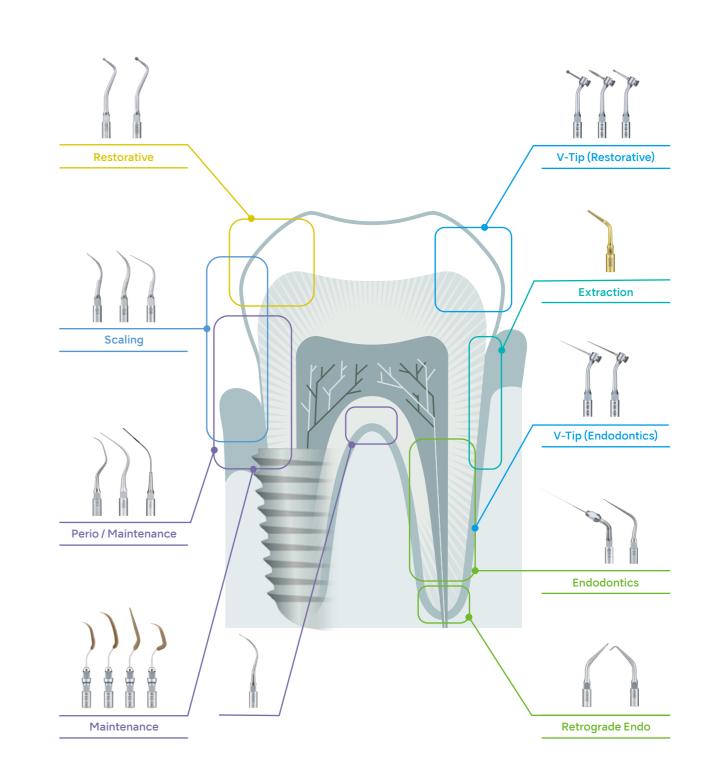

Light Probe



Gear Ratio	Optic	MODEL	ORDER CODE
	•	ISOL LOX	21203

• E-Type Optic Motor Connection • Stainless Steel Body • Glass Fiber Optics

 $[\breve{\mathcal{H}}]$ Washable in the thermodisinfector

Spare Parts and Accessories



VIVAMATE G5 MODEL PRODUCTS ORDER CODE MODEL PRO VIVAMATE G5 230 V U1068004 Handpiece Stand Control unit VMG30EM MC E1069002 Battery Pack VIVAMATE G5 Micromotor with Cord -FC-76M Z1082002 AC Adaptor Foot Control For VIVAMATE G5 ç

ORDER CODE

37064.6

37064.7

Z1035001

Z221076

VIVA ace

VIVAUCE								
MODEL	PRODUCTS	ORDER CODE	MODEL	PRODUCTS	ORDERCODE	MODEL	PRODUCTS	ORDERCODE
isoE-LUX		Z1265	V-ace Vacuum Junction Hose	6	U1144753	AC Power Cord	60	U439551
V-ace Vacuum Nozzle (Pack of 10)		U1146088	Vacuum Cap A))	U1144750	Shoulder Belt		U1144370
V-ace Vacuum Hose Se	et 🤤	U1144074	Vacuum Cap B		U1144751	Gasket For Water Bottle	\bigcirc	U1144729
V-ace Vacuum Bottle S	Set 📋.	U1146070	Hose Cap		U1144752	Gasket For Vacuum Bottle	\bigcirc	U1144636
V-ace Suction Waste Bottle		U1146075	V-ace Water Bottle Set		U1146080	Gasket For Vacuum Bottle	0	Z1137352
VS Suction Waste Bottle Lid		Z1240051	V-ace Water bottle		U1146085	Filter including O-Ring	۲	UA11440189
V-ace 3 way Syringe S	et O	U1144040	FC-76S Foot Control		Z1082005			
V-ace 3 way Syringe Nozzle	_	U1144046	FC-70S Foot Control	۲	Z1008006			

NAKANISHI INC. www.nsk-dental.com 700 Shimohinata, Kanuma, Tochigi 322-8666, Japan

Unit 12, 809-821 Botany Road, Rosebery NSW 2018, Australia

Specifications are subject to change without notice.

ODUCTS	ORDERCODE	MODEL	PRODUCTS	ORDER CODE
-	Z095205	AC Power Cord	60	U439551
	U1068052	Carrying Set For VIVAMATE G5	-	Y1002171
~	U1115			